

Best Allround Treatment for Hospitals




MOBILISATION SEAT

Support aid for early mobilisation

- + Safety for the patient
- + Relief for nursing care
- + Compliant with guidelines
- + Cost-efficient

MOBILISATION SEAT

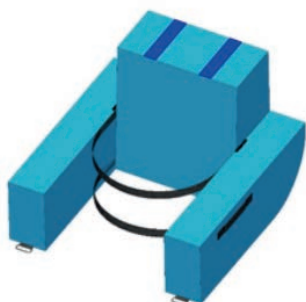

REF. 60.900.21.009 | The mobilisation seat



Adjustable headrest



Attachment to the hospital bed

The mobilisation seat is used as a support aid for the early mobilisation of patients. It is attached to an existing patient bed and can be customised to the patient. The armrests can be folded for easy handling. The included accessories can be customised for a wide range of hospital settings.

CONFIGURATION

- + 1 backrest 550 x 500 x 300 mm,
- + 2 armrests 800 x 270 x 150 mm (swivable)
- + 1 swivel unit
- + 1 headrest 380 x 300 x 300 mm (foldable)
- + 4 strap fasteners à 500 mm length
- + Abdominal and chest strap
- + Loops on the side for transport

- + Dimensions: 800 x 550/270 x 800 mm
- + Weight: 13 kg

MATERIAL

Combination core: 30 mm PE foam, firmly bonded to PU foam with integrated swivel unit.

Cover: PVC tarpaulin fabric with fastening and strap connection

The backrest is fitted with 2 velcro strips for attaching a headrest.

APPLICATION

1. Position the seat on the patient's bed. Fold down the armrests using the handles on the sides.
2. Ensure correct positioning and attach the seat to the bed frame using the integrated eyelets and the provided strap holders.
3. Securely stabilise the patient with the abdominal and chest strap.
4. If desired stabilisation of the head has not yet been achieved, the headrest can be attached to the backrest and adjusted according to the body height.



P. J. Dahlhausen & Co. GmbH
Adam-Riese-Straße 4
50996 Köln · Germany
Phone: +49 (0) 2236 39 13-0
Fax: +49 (0) 2236 39 13-409
export@dahlhausen.de
www.dahlhausen.de